


# Elos Accurate® Scan Body

## The story of the height difference between IO and IO SA

Elos Accurate® Scan Body for Nobel Conical Connection and NobelActive® are available in two versions - IO and IO SA - here's how to choose the right scan body depending on the situation.

***It is of utmost importance for the dental technician to choose the same version of scan body in the software as the one you scanned in reality.***

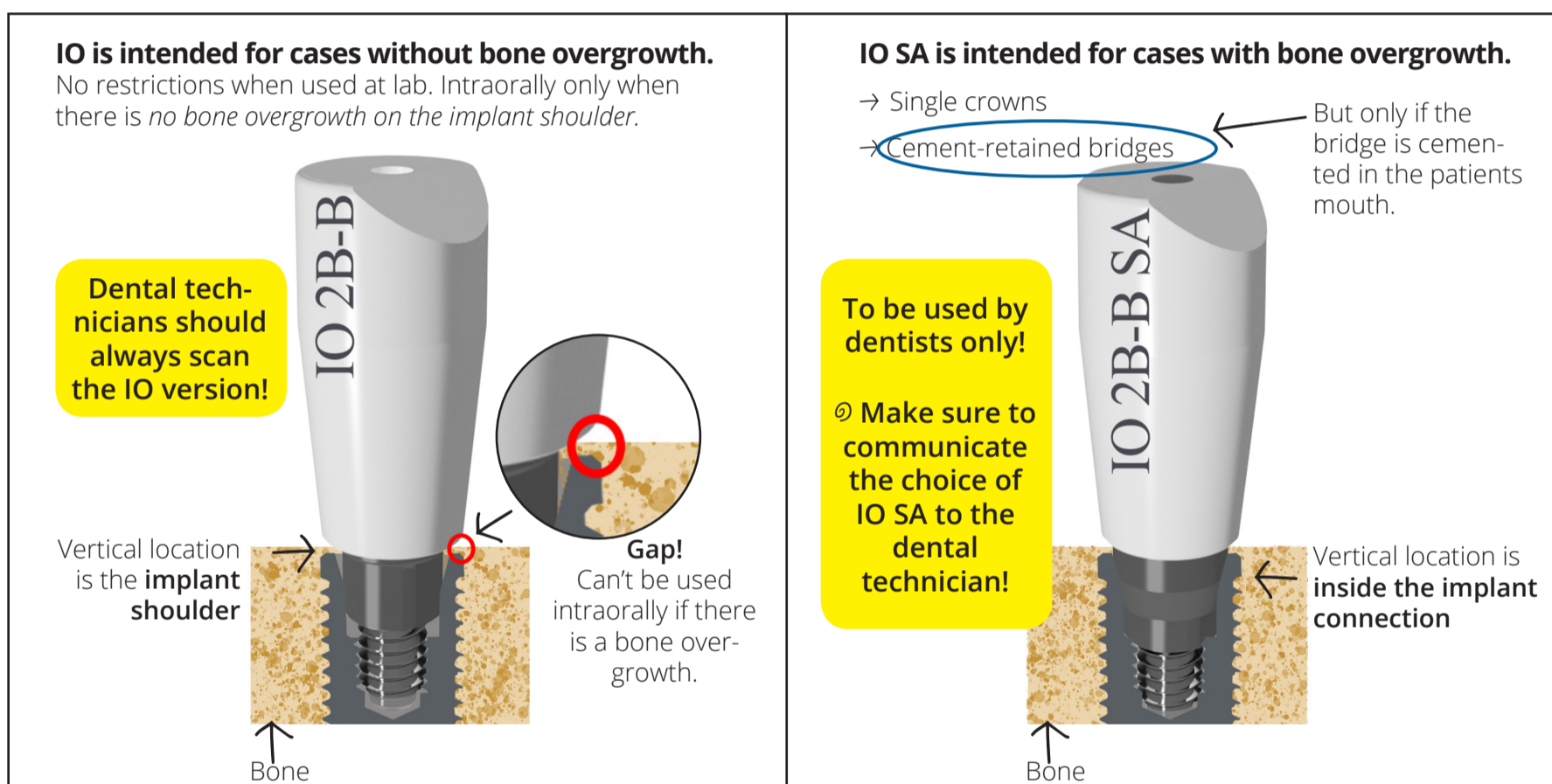

### Elos Accurate® Scan Body for Nobel Conical Connection and NobelActive®

**Material:** Titanium seating and PEEK in scanning area for optimal scanning results without spraying.

**Torque Value:** Hand-torque; no more than 5 Ncm.

